

Globasept[®]Wound Sol.

WOUND IRRIGATION SOLUTION

Globasept[®]Wound Sol. is suitable for cleaning, rinsing, moisturizing acute and chronic superficial wounds (incl. scratches, abrasions, bites, tears etc). Accelerates healing due to wound cleanliness. Wounds need enough moisture for optimal healing, as the skin cells must be able to divide and move for the healing process to take place. This is best possible in a humid wound environment.

PRODUCT PROPERTIES

- colourless
- does not sting or irritate
- suitable for both first aid and wound care
- also suitable for relieving insect bites and burns (including sunburn)

USAGE

Irrigate the required area evenly and completely.

Spray method (eg for wounds):

Spray the product abundantly over the entire area, making sure that the surface is evenly moistened.

Cleaning with a gauze:

Thoroughly moisten a sufficient amount of sterile gauzes and clean the required area completely.

Insect bites and sunburn:

Thoroughly moisten a sufficient amount of gauzes and clean the required area.

STORAGE

See the best before date on the package. Store at temperatures 5-25°C, keep away from direct sunlight.

UTILIZATION

The working solution is ecologically safe, all active ingredients are biodegradable in flush water. For recycling, rinse package with water.

WARNINGS!

The product is for external use only, not suitable for infusion or injection! Do not swallow! In very rare cases, a slight burning sensation may occur, but this dissipates after a few minutes. Globasept[®]Wound Sol. should not be used if you are allergic to any of the ingredients in this product. The product is not suitable for use in the middle or inner ear, hyaline cartilage or in the eyes. Avoid inhalation of aerosol. Do not use in combination with anionic surfactants as they may interact with polyhexanide. Due to insufficient clinical experience with pregnant and breastfeeding women, newborns and infants, the product should only be used after careful medical

evaluation. Do not use a product with damaged packaging. Bottles in contact with the wound must be discarded after use. If the wound does not begin to heal within a few days or if swelling, redness or pain develops in the wound area, consult a physician. Do not use consistently for longer than 1 month. Medical device - Keep out of reach of children.


COMPOSITION

100g of the product contains following ingredients: purified water, polyaminopropyl biguanide (polyhexanide) 0,1%


Product bears the CE marking according to Medical Device directive 93/42/EEC



PACKAGES

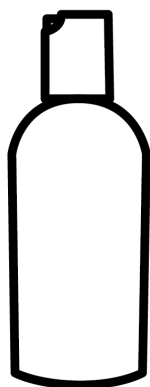

100ml flip-cap bottle

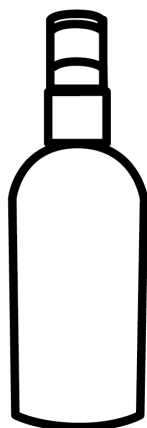

100ml spray bottle



Goodpoint Chemicals OÜ
Urda tee 3, Jälgimäe 76404
Saku vald, Harjumaa, Estonia
www.goodpointchemicals.com
info@goodpointchemicals.com

